

Clinical Data on MemoryGel™ Breast Implants



**Core Study: 3-Year
Cumulative Kaplan-Meier
Confirmed and Suspected
Rupture Risk Rates*
(MRI Cohort)¹**

**Core Study: 3-Year
Cumulative Kaplan-Meier
Confirmed Upon Explant
Rupture Risk Rates*
(MRI Cohort)¹**

**Core Study: 3-Year
Reported Symptomatic
Ruptures Only**
(Non-MRI Cohort)²**

Indication	Patient	Implant	Patient	Implant	Patient	Implant
Augmentation	0.5%	0.3%	0%	0%	0%	0%
Reconstruction	0.9%	0.5%	0%	0%	0%	0%
Revision-Augmentation	7.7%	5.9%	1.8%	1.8%	0%	0%
Revision-Reconstruction	0%	0%	0%	0%	0%	0%
Overall	1.6%	1.1%	0.2%	0.3%	0%	0%

¹ Sourced from Mentor's Core Gel Study Data

² Sourced from Mentor's SSED

*MRI Cohort N=420

**Non-MRI Cohort N=587

Please ensure that all patients receive a copy of *Important Information for Augmentation Patients about Mentor MemoryGel™ Silicone Gel-Filled Breast Implants* or *Important Information for Reconstruction Patients about Mentor MemoryGel™ Silicone Gel-Filled Breast Implants*. These educational brochures provide patients with information regarding the risks and benefits of breast implants. It is very important that your patient reads and fully understands the relevant brochure and has an opportunity to consult with you prior to deciding on surgery. Your patient should be advised to wait one to two weeks before scheduling surgery. For current contraindications, warnings, precautions, risks and complications for MemoryGel™ silicone gel breast implants, please refer to the Product Insert Data Sheet provided with each implant, or online at <http://www.mentorcorp.com/MemoryGel-PIDS>